



Thermal Pillow with Cherry Pits and Lavender Essence

C302AAC003



Heating pad with cherry stones and lavender aroma for a natural remedy against cramps and muscle tension.

The cushion, covered in cotton and divided into four sections, allows for even heat distribution for a uniform warming effect on the body. After heating in the microwave or oven, the cherry stone content releases heat, relieving pain and muscle stiffness. Quick and easy to use, with a removable cotton cover for washing. The soothing lavender essence is released when the cushion is heated.

Hot use: Heat the cushion inside in the oven (maximum 70°C for about 5 minutes) or in the microwave (800W for about 30 seconds); adjust the heating times according to different types of ovens. Reheat to the desired temperature and place on the body surface.

Cold use: Place the cushion in the refrigerator or freezer for approximately 30-45 minutes; remove and place on the desired body area.

Beper S.r.l.

Via Antonio Salieri, 30 37050 Vallesse di Oppeano (VR)

www.beper.com

beper



CALDO & FREDDO
HOT & COLD



FORNO & MICROONDE
OVEN & MICROWAVE



FRIGO
FRIDGE



LAVABILE
WASHABLE

Essenza di LAVANDA
LAVENDER essence



Thermal Pillow with Cherry Pits and Lavender Essence

C302AAC003

PRODUCT SIZE: 50X18X2 CM

GROSS WEIGHT: 1 KG KG

PACKAGING SIZE: 24X7X18 CM

PACKAGING: GIFT BOX

UNITS X PARCEL: 6



8056420226331

Beper S.r.l.

Via Antonio Salieri, 30 37050 Vallesse di Oppeano (VR)

www.beper.com