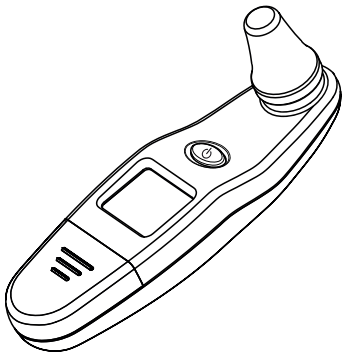
beper

- TERMOMETRO AURICOLARE DIGITALE
MANUALE DI ISTRUZIONI
- DIGITAL EAR THERMOMETER
USE INSTRUCTIONS
- THERMOMÈTRE AURICULAIRE DIGITAL
MANUEL D'INSTRUCTIONS
- DIGITALES OHR THERMOMETER
BETRIEBSANLEITUNG
- TERMOMETRO DIGITAL DE OÍDO
MANUAL DE INSTRUCCIONES
- ΨΗΦΙΑΚΟ ΘΕΡΜΟΜΕΤΡΟ ΑΥΤΙΟΥ
ΕΓΧΕΙΡΙΔΙΟ ΟΔΗΓΙΩΝ
- TERMOMETRU DIGITAL PENTRU URECHE
MANUAL DE INSTRUCȚIUNI

CE 0197

Cod.: 40.102
LOT 2018-132

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For further information and/or
for instructions for use in other languages,
please visit: **bepër.com**

Manufacturer:
Hangzhou Sejoy Electronics & Instruments Co., Ltd.
Building 2, No.202, Zhenzhong Road, West Lake
Economy & Technology Zone 310030 Hangzhou China
Website: www.sejoy.com
Email: info@sejoy.com
Telephone: +86-571-81957767
Fax: +86-571-81957750

The product is in compliance with the requirements of
MDD 93/42/EEC, "0197" is the identification number
of notify body;

European Authorized Representative:
Shanghai International Holding Corp. GmbH (Hamburg)
Eiffestrasse 80, 20537 Hamburg, Germany

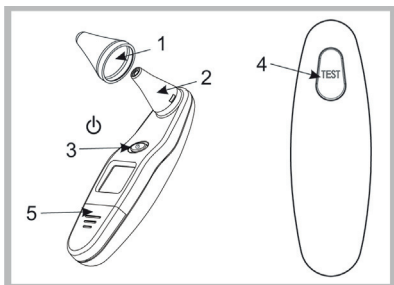
Fig.1

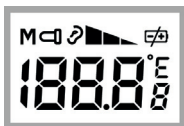
Fig.2

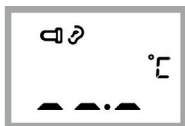
Fig.3

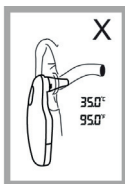
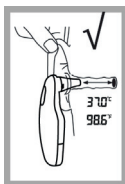
Fig.4

Fig.5

Fig.6

Fig.7

Fig.8

Fig.9

Fig.10

Fig.11

Fig.12

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GENERAL SAFETY ADVICE

READ THE OPERATING INSTRUCTIONS CAREFULLY BEFORE FIRST USE OF THE APPLIANCE.

Read the following instructions carefully, since they give you useful safety information about installation, use and maintenance and help you avoid mishaps and possible accidents.

Remove the packing and make sure that the appliance is intact, with special attention to the supply cable. The elements of the packing (plastic bags, polystyrene, etc) do not have to be accessible to the children, in order to preserve them from danger sources.

It is recommended not to throw these packing material in the domestic waste, but to deliver them to the appropriate station of collection or destruction refusals, asking eventually information to the manager of the Sanitation Department of your own municipality.

This appliance is to be destined to its special purpose only.

Any other use is to be considered as improper and thus dangerous. The manufacturer cannot be responsible for any damages deriving from improper, wrong or incautious use.

To avoid any heating risk, it is recommended to unroll completely the supply cord and to unplug the appliance, when not used.

To protect against electric shock, do not immerse cord, plug or any parts of the appliance into water or other fluids.

Do not hold the appliance with wet hands or feet.

Do not hold the appliance with naked feet.

Do not expose the appliance to atmospheric agents such as rain, wind, snow, hail.

Do not use the appliance near explosives, highly flammable materials, gases, burning flames, heaters.

DO NOT ALLOW CHILDREN, DISABLED PEOPLE OR EVERY PEOPLE WITHOUT EXPERIENCE OR TECHNICAL ABILITY TO USE THE APPLIANCE IF THEY ARE NOT PROPERLY WATCHED OVER. CLOSE SUPERVISION IS NECESSARY WHEN ANY APPLIANCE USED BY OR NEAR CHILDREN.

MAKE SURE THAT CHILDREN DO NOT PLAY WITH THIS APPLIANCE.

IF YOU DECIDE NOT TO USE THE APPLIANCE ANY LONGER, MAKE IT UNSERVICEABLE BY SIMPLY CUTTING THE SUPPLY CABLE AFTER DISCONNECTING THE PLUG FROM THE SOCKET.

ALSO MAKE ANY DANGEROUS PARTS UNSERVICEABLE, ESPECIALLY FOR CHILDREN WHO COULD PLAY WITH THE APPLIANCE.

Introduction

This digital ear thermometer is specifically designed for safe use in the eardrum. This digital ear thermometer is a device capable of measuring body temperature by detecting the intensity of infrared light emitted from the ear canal. It converts the measured heat into a temperature reading displayed on the screen. When properly used, the device will quickly assess body temperature in an accurate manner.

CAUTION! Please read instructions carefully and thoroughly before using the device

NOTE

- 1. This thermometer is intended for home use only, and it is not meant to replace a professional medical evaluation. Please consult your doctor if you have any concern about your health.**
- 2. There is no gender nor age limitation for using this device.**
- 3. To avoid inaccurate readings, always use a new, clean sensor cover for each temperature reading.**
- 4. Do not expose the thermometer to extreme temperatures (below $-25^{\circ}\text{C}/-13^{\circ}\text{F}$ or over $55^{\circ}\text{C}/131^{\circ}\text{F}$) nor excessive humidity ($>95\%RH$).**
- 5. This thermometer must only be used with original thermometer sensor covers.**
- 6. Keep the battery away from children.**

PRODUCT DESCRIPTION Fig.1

1. Sensor cover
2. Sensor
3. ON/OFF switch
4. TEST button
5. Battery cover

INSTRUCTIONS FOR USE

1. To achieve accurate readings, make sure a new, clean sensor cover is in place before each measurement. Place a new sensor cover on the sensor firmly, making sure the sensor and the inner portion of the sensor cover fit snugly. To ensure proper hygiene and to avoid contamination, a new sensor cover should be applied for each user.
2. Press the ON/OFF switch. The LCD screen will appear as shown in Figure 2. Wait for the acoustic signal and the screen to change to setting shown in Figure 3. The thermometer is now ready for a new measurement.

Note: The symbol  indicates that this thermometer must be only be used with original sensor covers, otherwise the accuracy of readings will degrade significantly.

3. Place the sensor snugly into the ear canal. Make sure the sensor tip faces the eardrum as shown in Figure 4. Press the TEST Button. The thermometer will emit a long beeping sound for approximately 1 second. When the beeping sound is over, the result of the measurement will be displayed on the screen.
4. The thermometer is ready for testing again once the ear symbol displays on the screen. A waiting period of 20 seconds between testing is recommended to ensure complete accuracy.
5. The thermometer will shut off automatically after 60 seconds of inactivity. To prolong battery life, press the ON/OFF switch to turn the device off.

TEMPERATURE MEASUREMENT HINTS

1. The temperature reading may differ from ear to ear, it is therefore advisable to always measure the temperature in the same ear.
2. Make sure that the ear is always free from obstructions or excessive earwax.
3. External factors may influence ear temperatures such as:
 - Laying your head on one ear or the other
 - Covering the ears for a long time
 - Having been exposed to very high or very low temperatures
 - Having been recently swimming or bathingIn these cases, remove the individual from the situation and wait 20 minutes before measuring the temperature.

4. People wearing hearing aids or ear plugs must remove the devices and wait 20 minutes before measuring the temperature.
5. When using the thermometer on infants under age of 1, pull the ear up making sure the sensor faces the eardrum. (See Figure 5)
6. When using the thermometer on individuals over the age of 1, pull the ear back making sure the sensor faces the eardrum. (See Figure 6)
7. Do not touch the sensor tip. A cotton swab and rubbing alcohol may be used to clean the sensor tip.
8. If thermometer is stored in a significantly different environment than testing location, place it in the testing location for approximately 30 minutes prior to use.

MEMORY MODE

1. The Memory Mode can be accessed in two ways:
 - a) After turning the thermometer on and waiting for the screen setting as shown in Figure 3, press and hold the ON/OFF switch. The letter "M" will appear in the upper left corner of the display. (See Figure 7)
 - b) After measuring the temperature, press and hold the ON/OFF switch for a few seconds until the letter "M" appears in the upper left corner of the screen.
2. The thermometer will automatically memorize the last 10 temperature readings. Each time the ON/OFF switch is pressed, the screen displays past readings that correspond with a number 1-0. The number 1 reflects the most recent reading while the number 0 reveals the oldest reading stored in memory. (See Figure 8)
3. In the memory mode, the ear symbol is always displayed on screen. To take a new measurement, simply press the TEST Button.

SETTING OF TEMPERATURE SCALE

1. Temperature readings are available in Celsius ($^{\circ}\text{C}$) or Fahrenheit ($^{\circ}\text{F}$) scale.
2. With the device off, press and hold the ON/OFF switch and continue holding until either $^{\circ}\text{C}$ or $^{\circ}\text{F}$ is displayed on screen as shown in Figure 9.
3. To change this setting to a different scale, release the ON/OFF switch and wait 1 second until screen displays data as shown in Figure 3.
4. Press the ON/OFF switch once and the device will turn off.

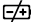
5. After the device has shut off, press and hold the ON/OFF switch again until the new scale appears.
6. Release the ON/OFF switch and wait 1 second, the unit is now ready for a new measurement in the selected scale.

CARE AND CLEANSING

1. Sensor covers are intended for single use only. Absent, damaged, perforated, or soiled sensor covers can lead to inaccurate temperature readings.
2. The sensor tip must be kept clean, dry, and undamaged at all times to ensure accurate readings. The accuracy of temperature readings can be affected by damage to the probe window, or the presence of dirt and ear wax on the probe window.
3. Fingerprints, earwax, dust and other soiling compounds reduce transparency of the sensor tip and result in lower temperature readings. If the thermometer is accidentally placed in the ear without a sensor cover installed, the tip must be cleaned immediately.
4. The sensor tip is the most delicate part of the thermometer. To safely clean the tip, gently wipe its surface with a cotton swab slightly moistened with isopropyl alcohol and immediately wipe dry with a clean cotton swab. After cleaning, allow at least 5 minutes drying time before measuring temperatures.
Note: Do not use any chemical other than isopropyl alcohol to clean the sensor tip.
5. Use a soft, dry cloth to clean the thermometer display and exterior.
6. The thermometer is not waterproof. Do not put the device in water when cleaning.
7. Store thermometer and sensor cover in a dry place, free from dust and contamination and away from direct sunlight.
8. Periodic cleaning and disinfection of the device following use are essential to prevent patient cross infection.
-Use a soft cloth slightly moistened with a 70% isopropyl alcohol solution to disinfect the thermometer and sensor. Do not use abrasive cleaners.
9. Ensure that children do not use the device unsupervised; some parts are small enough to be swallowed.
10. Strong electromagnetic fields may interfere with the proper operation of the thermometer.

Do not use this thermometer close to sources of strong electromagnetic radiation.

BATTERY REPLACEMENT

1. Replace battery when the symbol  appears in the upper right corner of LCD screen. (See Figure 10)
2. Slide battery cover down as shown in Figure 11.
3. Remove battery as shown in Figure 12.
4. Replace with one new battery(CR 2032) with "+" side facing up.
5. Dispose exhausted battery according to applicable local laws and regulations.
6. Slide battery cover back on.

TECHNICAL DATA

Measuring range	32.0°C - 43.0°C (89.6°F - 109.4°F)
Reference body site	Oral (This thermometer converts the ear temperature to display its "oral equivalent.")
Operation mode	Adjusted mode
Laboratory accuracy	±0.2 °C (0.4 °F) all'interno del range 35.0~42.0°C (95.0°F~107.6°F), con temperatura di funzionamento compresa tra 15°C e 35°C (59.0°F~95.0°F) ±0.3 °C (0.5°F) per altri range di temperatura
Operating temperature range	±0.2°C (0.4°F) during 35.0°C - 42.0°C (95.0°F - 107.6°F) at 15°C - 35°C (59.0°F - 95.0°F) operating temperature range ±0.3°C (0.5°F) for other measuring and operating temperature range

Storage and transport	10°C – 40°C (50°F – 104°F), 15%~85%RH, non-condensing
temperature range	-25°C – 55°C (-13°F – 131°F), 15%~95%RH, non-condensing
Battery	One CR2032 lithium cell battery
Battery life	Approx. 1 year/3000 readings

For any improvement reasons, Beper reserves the right to modify or improve the product without any prior notice.

TROUBLESHOOTING

Error message	Problem	Solution
Er 1	Measurement taken before thermometer is ready	Wait until the symbol  appears on screen, then take a new measurement
Er 2	Environment temperature is outside the range between 10 °C and 40°C (50°F – 104°F)	Place the thermometer in a place with room temperature between 10 °C and 40°C (50°F – 104°F) for at least 30 minutes

Error message	Problem	Solution
Er3	The thermometer is placed incorrectly or unsteady	Read instructions for use thoroughly and take a new measurement
Er4	The thermometer is detecting a rapid environment temperature change	Allow the thermometer to rest in a place with room temperature between 10 °C and 40°C (50°F – 104°F) for at least 30 minutes
Er5	The thermometer is not functioning properly	Take the battery out of the device, wait for 1 minute and put the battery back on. If the message reappears, contact the supplier for service
H₀	Temperature measured is higher than 43.0 °C (109.4 °F)	Read instructions for use thoroughly, then check the integrity of the sensor cover and take a new measurement
Lo	Temperature measured is lower than 32.0°C (89.6 °F)	Read instructions for use thoroughly, then check the integrity of the sensor cover and take a new measurement

Error message	Problem	Solution
	Battery is dead	Replace battery with a new CR2032 battery

GUARANTEE CERTIFICATE

This appliance has been checked in the factory. From the date of original purchase a 24 month guarantee applies to material and production defects. The purchase receipt and the guarantee certificate must be submitted together in case of claim to guarantee.

The guarantee is valid only with a guaranty certificate and proof of purchase (fiscal receipt) indicating the date of purchase and the model of the appliance

For any technical assistance, please contact directly the seller or our head office in order to preserve the appliance's efficiency and NOT TO void the guarantee. Any intervention on this appliance by non-authorized persons will automatically void the guarantee.

GUARANTEE CONDITIONS

If the appliance shows defects as a result of faulty material and/or production during the period of guarantee, we guarantee repair free of charge on condition that:

- The appliance has been used properly and for the purpose for which it has been intended.
- The appliance has not been tampered with, otherwise it is not maintainable.
- The purchase receipt shall be presented.
- The appliance showing a fair wear and tear shall not be covered by this guarantee.

Therefore, any part that could be accidentally broken or having visible signs of use in consumable products (such as lamps, batteries, heating elements...) the aesthetic parts are excluded from the guarantee, and any defect whatsoever resulting from non-respect of the rules for use, negligence in usage and/or maintenance of the appliance, carelessness, wrong or improper installation, damage during transport and any other damage not attributable the supplier.

For every defect that could not be repaired within the guarantee period, the appliance will be replaced free of charge.

In any case, if the part to be replaced for defect, breakage or malfunction is an accessory and/or a detachable part of the product, Beper reserves the right to replace only the very part in question and not the entire product. For technical assistance and/or repairs after the guarantee period you can directly contact the below address: CONTACT YOUR DISTRIBUTOR IN YOUR COUNTRY OR ATER SALES DEPARTMENT BEPER. E-MAIL assistenza@beper.com WHICH WILL FORWARD YOUR ENQUIRIES TO YOUR DISTRIBUTOR.

The European directive 2011/65/EU on Waste Electrical and Electronic Equipment (WEEE), requires that old household electrical appliances must not be disposed of in the normal unsorted municipal waste stream. Old appliances must be collected separately in order to optimize the recovery and recycling of the materials they contain, and reduce the impact on human health and the environment. The crossed out "wheeled bin" symbol on the product reminds you of your obligation, that when you dispose of the appliance, it must be separately collected.

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