# *beper*

IT	MISURATORE DI PRESSIONE DA BRACCIO	pag. 4
EN	ARM BLOOD PRESSURE MONITOR	pag. 11
FR	MANOMÈTRE À BRAS	pag. 18
DE	ARMDRUCKMESSGERÄT	pag. 24
ES	TENSIÓMETRO DE BRAZO	pag. 31
GR	ΠΙΕΣΟΜΕΤΡΟ	pag. 38
RO	TENSIOMETRU DE BRAȚ	pag. 45
CZ	MONITOR KREVNÍHO TLAKU PAŽE	pag. 52
NL	BLOEDDRUKMETER VOOR DE ARM	pag. 60
LV	ROKU ASINSSPIEDIENA MĒRĪTĀJS	pag. 67

For further information and/or for instructions for use in other languages, please visit: [beper.com](http://beper.com)

Manufacturer:  
JOYTECH HEALTHCARE CO. LTD.  
No.365,Wuzhou Road,Yuhang Economic Development Zone,  
hangzhou city,zhejiang province 311100, China

European Authorized Representative:  
Shanghai International Holding Corp. GmbH (Europe)  
Eiffestrasse 80, 20537 Hamburg, Germany

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**assistenza@beper.com**

**Safety Notice**

For specific information about your blood pressure, please consult your doctor.

**Explanation of symbols:**

	Please read this manual thoroughly before using the unit. Please retain this manual for future reference Operate the unit only as intended.
	Caution: Do not mix new and old batteries simultaneously. Do not mix batteries of different brands. Install batteries paying attention to polarity indicated. Remove batteries from the device in case you will not use for long time. Replace batteries when on the display will appear this symbol  Dispose batteries properly, observe local laws and regulation.
	Appliance type B
	Factory
	Forbidden: Do not allow children and / or persons with reduced capacity to intend to use this unit. Do not attempt to repair the product yourself. When using this product do not use mobile phones or other devices that may generate electromagnetic fields.
	Mandatory: Contact your doctor if the results are regularly abnormal. Use the appliance only as described.
	Authorized representative in the European community

## Arm Blood Pressure Monitor Use Instructions

### Product description Fig. A

1. Inlet for air tube
2. WHO (World Health Organisation) blood pressure classification indicator
3. Display
4. Systolic blood pressure
5. Diastolic blood pressure
6. Pulse rate
7. Mode button
8. On/ Off switch button
9. Set button
10. Battery compartment
11. Arm cuff

### Display description Fig. B

1. Groups
2. Low batteries indicator
3. WHO (World Health Organisation) blood pressure classification indicator
4. Average of last 3 readings
5. Irregular pulse indicator
6. Time
7. Date
8. Pulse rate
9. Diastolic blood pressure
10. Systolic blood pressure

### Some important guidelines

Avoid eating, exercising, and bathing for 30 minutes prior to testing.

Sit in a calm environment for at least 5 minutes prior to testing.

Do not stand while testing.

During the testing, sit in a relaxed position while keeping your wrist level with your heart.

Avoid speaking or moving body parts while testing.

Wait 3 minutes or longer before re-testing.

It is suggested to measure your blood pressure at the same time, on same wrist, in the same position, and possibly in the same position.

This blood pressure monitor is not recommended for people with severe arrhythmia.

### Batteries installation Fig. A10

Slide off battery cover as indicated by the arrow, install 4 AAA batteries according to polarity, close battery cover.

### System Settings

With power off, press "SET" button to activate System Settings. The memory group icon flashes.

Select memory group

In the system setting, you may accumulate test results into 2 different groups. This allows multiple users to save individual test results (up to 60 memories per group.) Press "MODE" button to choose a group. The results of the tests will automatically save in each selected group.

**Time /date setting**

Press "SET" button again to set the time/date. First, set the month by pressing the "MODE" button. Press "SET" button again to confirm current month. Continue setting the day, hour, and minute in the same way. Every time the "SET" button is pressed, it will lock in your selection and continue in succession (month, day, hour, minute.)

**Settings saved**

In any setting mode, press "ON/OFF" button to turn off the unit

All information will be saved.

If unit is switch on and not in use for 3 minutes, it will automatically save all information and shut off.

**Use**

After installing the batteries and setting up the system, apply the arm cuff, inserting the pin located on the plastic tube of the arm cuff in the appropriate inlet on the main unit. Fig. C

Remove clothing from the arm.

Rest for several minutes before the test. Sit in a quiet place, preferably at a desk or table, with the arm resting on a stable surface and feet flat on the floor. Fig. D

Apply the cuff to the left arm and keep it at heart level. The bottom of the cuff should be placed approximately 1-2cm above the elbow. Fig. E – F

At this point you can proceed with the test.

Press and hold "ON/OFF" button until a beep sounds. On the LCD display will appear for one second as unit performs a quick diagnosis. A long tone indicates device is ready for testing.

**Note: Unit will not function if residual air from previous testing is present in cuff.**

On the LCD will flash an arrow until pressure is stabilized.

Initial pressure is first pumped to 190 mmHg. If the current user's systolic blood pressure is over 190mmHg, the unit will automatically re-inflate to the proper grade.

**Note: Pressurization will gradually subside and ultimately stop when cuff is not properly applied to the arm. If this occurs, press "ON/OFF" button to turn the unit off.**

After cuff inflation, air will slowly subside as indicated by the corresponding cuff pressure value. A heart symbol flashing will appear simultaneously on screen showing heart beat detection.

Safety Precaution: If pressure in arm cuff becomes too extreme while testing, press the "ON/OFF" button to turn power off. The cuff pressure will rapidly dissipate once the unit is off.

**Note: keep relaxed during testing. Avoid speaking or moving body parts.**

Three short beeps sound when testing is complete.

The screen will display measurements for systolic and diastolic blood pressure.

An indicator representing the current measurement will appear next to the corresponding WHO Classification.

After reading the test results, turn off the appliance by using the appropriate button, in case no button will be pressed the unit will turn off itself after about 3 minutes

**Note: make sure to select the appropriate memory group before testing.**

If the number of tests surpasses the allotted 60 memories per group, the most recent tests will appear first, thus eliminating the oldest readings.

**Note: Do not be alarmed if an abnormal reading occurs. A better indication of an individual's blood pressure occurs after 2-3 readings are taken at the same time each day over an extended period of time. Consult your doctor if test results remain abnormal.**

If the symbol  should appear on the display and four beeps are heard it means that the batteries life is depleting and need to be replaced.

Should the word "Err" appear on the display, it could be that the cuff has not been positioned correctly or that the blood pressurization is above 300mmHg. Try to adjust the cuff and/or relax then try again.

#### **Irregular heartbeat indicator**

If the monitor detects an irregular heart rhythm two or more times during the measurement process, a heart symbol appears, representing the irregular heartbeat, along with measurement results. Irregular heart rhythm is defined as the rhythm that is 25% slower or faster than the average rhythm detected during the measurement of systolic and diastolic blood pressure.

Consult your doctor if the irregular heartbeat symbol appears often during test results.

**Note: Make sure you have selected the appropriate memory group before the test.**

#### **Consultation and clearing of memory**

With the appliance switched off, it is possible to check the results of the passed tests with the "MODE" button. The last and oldest test result in memory can be shown by pressing and holding the "MODE" button. While reading the test results, you can press the "MODE" button to scroll through all the stored test results.

**Note: The results of previous tests will be displayed only by the most recently used memory group. To check the results of previous tests of other memory groups, you must first select the desired group and then turn off the monitor.**

The memory of the selected group can be cancelled in memory check mode. Press and hold the "SET" button for about 3 seconds to delete all the memory results from the selected group. The monitor emits an acoustic signal indicating cancellation and then goes directly to test mode. Press the "ON/OFF" button to turn off the unit.

**Note: Memory can not be recovered once it has been deleted.**

#### **Average of last 3 tests**

With the appliance power off, press and hold the "MODE" button to activate screen display. The unit performs a self-diagnosis, the screen will display the average test results from the last 3 readings of the last group used. The "AVG" symbol will appear along with the corresponding WHO blood pressure indicator. The memory check mode can be accessed by pressing the "MODE" button. To check the average results from other groups, select the desired group prior to activate the "MODE" button with the appliance powered off.

**WHO blood pressure classification indicator**

The appliance is equipped with a classification indicator based on established guidelines from the World Health Organization (WHO).

WHO classification	Systolic (mmHg)	Diastolic (mmHg)
Hypertension grade 3	≥180	≥110
Hypertension grade 2	160-179	100-109
Hypertension grade 1	140-159	90-99
High – normal	130-139	85-89
Normal	120-129	80-84
Optimal	<120	<80

**Maintenance**

Avoid dropping, slamming, or throwing the unit.

Avoid extreme temperatures. Do not expose unit directly under sunshine.

When cleaning the unit, use a soft fabric lightly dampened.

To clean the cuff apply a small amount of rubbing alcohol to a soft cloth, avoid to let liquids penetration in the appliance. Dry thoroughly before storing the product.

Remove the batteries when not in use for a prolonged period of time.

It is recommended to check the performance at least every 2 years.

## Arm Blood Pressure Monitor Use Instructions

Product description	Arm blood pressure monitor
Model number	40.120
Display	LCD digital - size: 66mm x 43 mm
Measuring method	Oscillometric
Measurement intervals	Pressure : 0 mmHg - 300 mmHg Pulsations: from 30 to 180 beats per minute
Measurement accuracy	Pressure: $\pm 3$ mm/Hg Pulsations : $\pm 5\%$
Pressurization	Automatic
Memory	120 memories with date and time
Functions	Low battery indicator WHO classification indicator Auto shut off Irregular heart rate indicator Average last 3 readings
Power supply	4 batteries type AAA
Batteries life	approx. 2 months based on 3 measurements per day
Unit weight	approx. 405 g (batteries not included)
Size	139 x 88 x 42 mm approx.
Arm circumference	Adjustable circumference 22 - 36cm
Protection Rating	IP22
Use environment conditions	Temperature: 10°C <sup>m</sup> -40°C (50°F - 104°F) Humidity: 15%-90% RH Atmospheric pressure
Storage environment conditions	Temperature: -20°C -55°C Humidity: $\leq 93\%$ RH

### Directives and rules:

This product meets the requirements of the EU standard for non-invasive medical devices. It has been certified according to EC directives and is provided with the CE mark (conformity mark) "CE 0197". The product meets the European standards EN 60601-1, EN 60601-1-2, EN 60601-1-11, EN 80601-2, EN 1060-1, EN 1060-3 and EN 1060-4. The appliance meets the requirements of the Council Directive "93/42 / EEC of 14 June 1993 on medical devices

The European directive 2011/65/EU on Waste Electrical and Electronic Equipment (WEEE), requires that old household electrical appliances must not be disposed of in the normal unsorted municipal waste stream. Old appliances must be collected separately in order to optimize the recovery and recycling of the materials they contain, and reduce the impact on human health and the environment. The crossed out "wheeled bin" symbol on the product reminds you of your obligation, that when you dispose of the appliance, it must be separately collected.

### **GUARANTEE CERTIFICATE**

This appliance has been checked in the factory. From the date of original purchase a 24 month guarantee applies to material and production defects. The purchase receipt and the guarantee certificate must be submitted together in case of claim to guaranteee.

### **The guarantee is valid only with a guaranty certificate and proof of purchase (fiscal receipt) indicating the date of purchase and the model of the appliance**

For any technical assistance, please contact directly the seller or our head office in order to preserve the appliance's efficiency and NOT TO void the guarantee. Any intervention on this appliance by non-authorized persons will automatically void the guarantee.

### **GUARANTEE CONDITIONS**

If the appliance shows defects as a result of faulty material and/or production during the period of guarantee, we guarantee repair free of charge on condition that:

- The appliance has been used properly and for the purpose for which it has been intended.
- The appliance has not been tampered with, otherwise it is not maintainable.
- The purchase receipt shall be presented.
- The appliance showing a fair wear and tear shall not be covered by this guarantee.

Therefore, any part that could be accidentally broken or having visible signs of use in consumable products (such as lamps, batteries, heating elements...) the aesthetic parts are excluded from the guarantee, and any defect whatsoever resulting from non-respect of the rules for use, negligence in usage and/or maintenance of the appliance, carelessness, wrong or improper installation, damage during transport and any other damage not attributable the supplier.

For every defect that could not be repaired within the guarantee period, the appliance will be replaced free of charge.

In any case, if the part to be replaced for defect, breakage or malfunction is an accessory and/or a detachable part of the product, Beper reserves the right to replace only the very part in question and not the entire product

Contact your distributor in your country or ater sales department Beper.

E-mail [assistenza@beper.com](mailto:assistenza@beper.com) which will forward your enquiries to your distributor.

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