beper

IT	MISURATORE DI PRESSIONE DA POLSO	pag. 5
EN	WRIST BLOOD PRESSURE MONITOR	pag. 13
FR	TENSIO MÈTRE AU POIGNET	pag. 21
DE	BLUTDRUCK-MESSGERÄT FÜR DAS HANDGELENK	pag. 30
ES	TENSIÓMETRO DE MUÑECA	pag. 39
GR	ΠΙΕΣΟΜΕΤΡΟ ΚΑΡΠΟΥ	pag. 47
RO	TENSIOMETRU DE ÎNCHEIETURĂ	pag. 56
CZ	MONITOR KREVŇÍHO TLAKU NA ZÁPĚSTÍ	pag. 64
NL	BLOEDDRUKMETER VOOR DE POLS	pag. 73
PL	CIŚNIENIOMIERZ NADGARSTKOWY	pag. 81
LV	PLAUKSTAS LOCĪTAVAS ASINSSPIEDIENA MĒRĪTĀJS	pag. 89

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assistenza@beper.com

Safety Notice

For specific information about your blood pressure, please consult your doctor.

Explanation of symbols

	Please read this manual thoroughly before using the unit. Please retain this manual for future reference Operate the unit only as intended.
	Caution: Do not mix new and old batteries simultaneously. Do not mix batteries of different brands . Install batteries paying attention to polarity indicated. Remove batteries from the device in case you will not use for long time. Replace batteries when on the display will appear this symbol  Dispose batteries properly, observe local laws and regulation.
	Appliance type BF
	Factory
	Forbidden: Do not allow children and / or persons with reduced capacity to intend to use this unit. Do not attempt to repair the product yourself. When using this product do not use mobile phones or other devices that may generate electromagnetic fields.
	Mandatory: Contact your doctor if the results are regularly abnormal. Use the appliance only as described.
	Authorized representative in the European community

Product description Fig. A

1. WHO (World Health Organisation) blood pressure classification indicator
2. Display
3. Systolic blood pressure
4. Diastolic blood pressure
5. Pulse rate
6. On/ Off switch button 
7. "M" button
8. Wrist cuff

Display description Fig. B

1. Low batteries indicator
2. WHO (World Health Organisation) blood pressure classification indicator
3. Time
4. Date
5. Systolic blood pressure
6. Diastolic blood pressure
7. Pulse rate

Some important guidelines

Avoid eating, exercising, and bathing for 30 minutes prior to testing.

Sit in a calm environment for at least 5 minutes prior to testing.

Do not stand while testing.

During the testing, sit in a relaxed position while keeping your wrist level with your heart.

Avoid speaking or moving body parts while testing.

Wait 3 minutes or longer before re-testing.

It is suggested to measure your blood pressure at the same time, on same wrist, in the same position, and possibly in the same position.

This blood pressure monitor is not recommended for people with severe arrhythmia.

Time and date setting

With the appliance powered off, but with batteries installed, press  button 3 seconds. Set the month first by adjusting the "M" button. Press  button again to confirm current month. Continue setting the day, hour, and minute with the same method.

Every time the  button is pressed, it will lock in your selection and continue in succession (month, day, hour, minute.)

To save settings:

While in any setting mode, press  button about 3seconds to turn the unit off.

All information will be saved.

Note: Unit will automatically save all information and shut off if left it for 3 minutes.

Use

Install batteries by sliding battery cover off as indicated by arrow, install two AAA batteries according to polarity, close battery cover. Fig. C

Free the wrist from any clothing. Do not apply the appliance over clothing. Fig. D

Apply the cuff to the left wrist and hold it at the same level as the heart level. Place it about 1-2 cm from your wrist. Fig. E

Note: Pressurization will gradually subside and ultimately stop when cuff is not properly applied to the wrist.

If this occurs, pressing the  button to turn the unit off.

Safety Precaution: If pressure in cuff becomes too extreme while testing, press the  button to turn power off.

The cuff pressure will rapidly dissipate once the unit is off.

To start the testing, press and hold  button until a beep sounds. Fig. F. On the LCD display will appear for one second as unit performs a quick diagnosis.

Note: Unit will not function if residual air from previous testing is present in cuff.

On the display will flash an arrow until pressure is stabilized.

Initial pressure is first pumped to 190 mmHg. If the current user's systolic blood pressure is over 190mmHg, the unit will automatically re-inflate to the proper shelf. After cuff inflation, air will slowly subside, simultaneously on the display the heart symbol will flash meaning that heart beat detection is testing.

Note: During testing it is advisable to remain relaxed, avoid speaking or moving body parts.

Three short beeps sound when testing is complete. The display will display measurements results.

After reading the test results, turn off the appliance by using the appropriate button, in case no button will be pressed the unit will turn off itself after about 3 minutes.

Note: Do not be alarmed if an abnormal reading occurs. A better indication of an individual's blood pressure occurs after 2-3 readings are taken at the same time each day over an extended period of time. Consult your doctor if test results remain abnormal.

If the symbol  should appears on the display and four beeps are heard it means that the batteries life is depleting and need to be replaced.

Should the word "Err" appear on the display, it could be that the cuff has not been positioned correctly or that the blood pressurization is above 300mmHg. Try to adjust the cuff and/or relax then try again.

The product is equipped with a memory, with the appliance switched off, it is possible to check the old test results by pressing the "M" key.

The results memory can be deleted in memory check mode. Press and hold the  button for about 3 seconds to delete all memory results. The monitor emits an acoustic signal indicating the cancellation and then goes directly to test mode. Press the  button to turn off the unit.

Note: The memory cannot be recovered once it has been deleted.

WHO blood pressure classification indicator

The appliance is equipped with a classification indicator based on established guidelines from the World Health Organization (WHO).

WHO classification	Systolic (mmHg)	Diastolic (mmHg)
Hypertension grade 3	≥180	≥110
Hypertension grade 2	160-179	100-109
Hypertension grade 1	140-159	90-99
High – normal	130-139	85-89
Normal	120-129	80-84
Optimal	<120	<80

Maintenance

Avoid dropping, slamming, or throwing the unit.

Avoid extreme temperatures. Do not expose unit directly under sunshine.

When cleaning the unit, use a soft fabric lightly damped.

To clean the cuff apply a small amount of rubbing alcohol to a soft cloth, avoid to let liquids penetration in the appliance. Dry thoroughly before storing the product.

Remove the batteries when not in use for a prolonged period of time.

It is recommended to check the performance at least every 2 years.

Product description	Wrist blood pressure monitor
Model number	40.121
Display	LCD digital - size: 45mm x 30 mm
Measuring method	Oscillometric
Measurement intervals	Pressure : 0 mmHg - 300 mmHg Pulsations: from 30 to 180 beats per minute
Measurement accuracy	Pressure: ± 3 mm/Hg Pulsations : $\pm 5\%$
Pressurization	Automatic
Memory	120 memories with date and time
Functions	Low battery indicator WHO classification indicator Auto shut off
Power supply	2 batteries type AAA
Batteries life	approx. 2 months based on 3 measurements per day
Unit weight	approx. 110 g (8 oz.) (batteries not included)
Size	79 x 66 x 28 mm approx.
Arm circumference	ca. 77 x 300 mm Adjustable circumference 13,5 - 21,5 cm
Protection Rating	Protection Rating IP 22

Use environment conditions	Temperature: 10°C~40°C (50°F - 104°F) Humidity: 15%~93% RH Pressure: 700hPa~1060 hPa
Storage environment conditions	Temperature: -25°C ~70°C (-13°F~158°F) Humidity: ≤ 93% RH

Directives and rules

This product meets the requirements of the EU standard for non-invasive medical devices. It has been certified according to EC directives and is provided with the CE mark (conformity mark) "CE 0197". The product meets the European standards EN 60601-1, EN 60601-1-2, EN 60601-1-11, EN 80601-2, EN 1060-1, EN 1060-3 and EN 1060-4. The appliance meets the requirements of the Council Directive "93/42 / EEC of 14 June 1993 on medical devices.

For any improvement reasons, Beper reserves the right to modify or improve the product without any notice.

The European directive 2011/65/EU on Waste Electrical and Electronic Equipment (WEEE), requires that old household electrical appliances must not be disposed of in the normal unsorted municipal waste stream. Old appliances must be collected separately in order to optimize the recovery and recycling of the materials they contain, and reduce the impact on human health and the environment. The crossed out "wheeled bin" symbol on the product reminds you of your obligation, that when you dispose of the appliance, it must be separately collected.

GUARANTEE CERTIFICATE

This appliance has been checked in the factory. From the date of original purchase a 24 month guarantee applies to material and production defects. The purchase receipt and the guarantee certificate must be submitted together in case of claim to guarantee.

The guarantee is valid only with a guaranty certificate and proof of purchase (fiscal receipt) indicating the date of purchase and the model of the appliance

For any technical assistance, please contact directly the seller or our head office in order to preserve the appliance's efficiency and NOT TO void the guarantee. Any intervention on this appliance by non-authorized persons will automatically void the guarantee.

GUARANTEE CONDITIONS

If the appliance shows defects as a result of faulty material and/or production during the period of guarantee, we guarantee repair free of charge on condition that:

- The appliance has been used properly and for the purpose for which it has been intended.
- The appliance has not been tampered with, otherwise it is not maintainable.
- The purchase receipt shall be presented.
- The appliance showing a fair wear and tear shall not be covered by this guarantee.

Therefore, any part that could be accidentally broken or having visible signs of use in consumable products (such as lamps, batteries, heating elements...) the aesthetic parts are excluded from the guarantee, and any defect whatsoever resulting from non-respect of the rules for use, negligence in usage and/or maintenance of the appliance, carelessness, wrong or improper installation, damage during transport and any other damage not attributable the supplier.

For every defect that could not be repaired within the guarantee period, the appliance will be replaced free of charge.

In any case, if the part to be replaced for defect, breakage or malfunction is an accessory and/or a detachable part of the product, Beper reserves the right to replace only the very part in question and not the entire product

Contact your distributor in your country or after sales department Beper.

E-mail assistenza@beper.com which will forward your enquiries to your distributor.

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