
- NEBULIZZATORE - MANUALE DI ISTRUZIONI
 - NEBULIZER - USE INSTRUCTIONS
- NÉBULISEUR - MANUEL D'INSTRUCTIONS
 - INHALATOR - BETRIEBSANLEITUNG
- NEBULIZADOR - MANUAL DE INSTRUCCIONES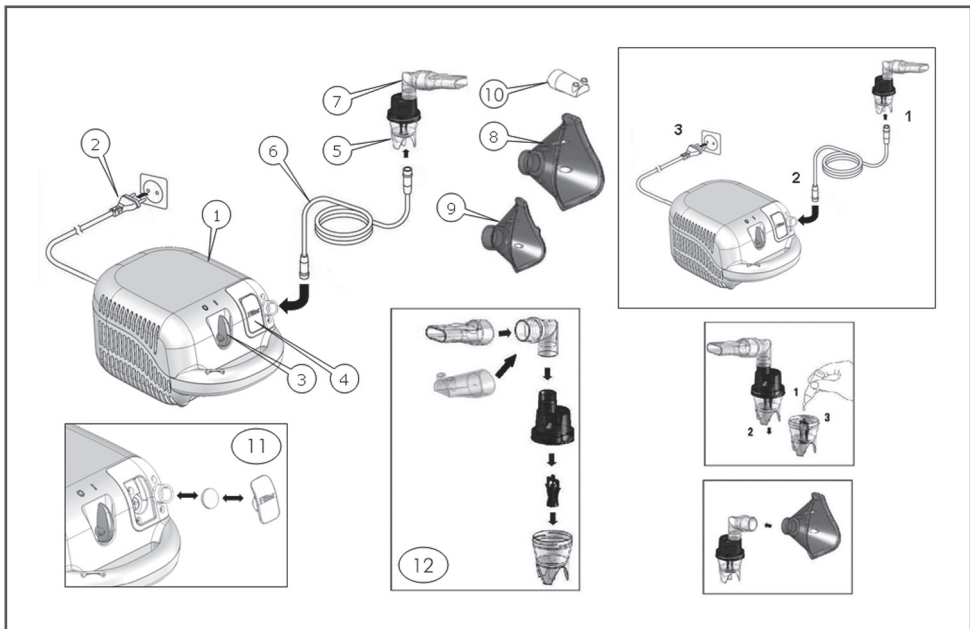

Cod.: 40.110

ITALIANO	pag. 3
ENGLISH	pag. 7
FRANÇAIS	pag. 10
DEUTSCH	pag. 14
ESPAÑOL	pag. 18

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Tel. 045/7134674 – Fax 045/6984019

e-mail: assistenza@beper.com

PROFESSIONAL PISTON COMPRESSOR NEBULISER

- | | | | | | | | |
|---|-------------------|---|------------------------|---|-----------------|----|--------------------------|
| 1 | Piston compressor | 4 | Air filter compartment | 7 | Mouthpiece | 10 | Nosepiece |
| 2 | Power cord | 5 | Nebjet Nebulizer | 8 | Adult face mask | 11 | Replacing air filter |
| 3 | ON/OFF Switch | 6 | Air hose | 9 | Child face mask | 12 | Assembling nebuliser kit |

Dear Customer,

this nebuliser is a high-quality device for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses. This medical device generates aerosols with variable characteristics to adapt to the patient's respiratory pathology and generating a medicinal mist of particles that are small enough to reach even the deepest regions of your lungs and be of maximal benefit. The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy. Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your Beper product. For any further questions you may have, please ask your chemist or usual retailer.

IMPORTANT SAFETY INSTRUCTIONS

This device may only be used for the purposes described in this booklet. The manufacturer cannot be held liable for damage caused by incorrect application. This device is not suitable for anaesthesia and lung ventilation. This device should only be used with original accessories as shown in these instructions. Do not use the device if you think it is damaged or notice anything unusual. Never open the device. This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section. Protect it from: - water and moisture - extreme temperatures - impact and dropping - contamination and dust - direct sunlight - heat and cold. Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy. All accessories should only be used by one person. Never bend the nebuliser over 60°.

- The unit should not be used in the presence of anesthetic mixture inflammable with air, oxygen or nitrous oxide.
- Device not protected against splashes

 Ensure that children do not use the device unsupervised some parts are small enough to be swallowed.

 Use of this device is not intended as a substitute for consultation with your physician.

In view of their length, the power cord and connection hose could constitute a strangulation hazard.

PREPARATION AND USAGE OF THIS DEVICE

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

1 Assemble the nebuliser kit **12** Ensure that all parts are complete.

2 Fill the nebuliser with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.

3 Connect the nebuliser with the air hose **6** to the compressor **1** and plug the power cord **2** into the socket. **4.** Switch the ON/OFF switch **3** to position «**I**» to turn on the device and place the mouthpiece **7** in the mouth or fit one of the face masks over mouth and nose. The mouthpiece gives you a better drug delivery to the lungs. Choose between adult **8** or child face mask **9** and make sure that it encloses the mouth and nose area completely. **5.** Breathe in and out calmly during the therapy. Sit in a relaxed position with the upper body upright. **Do not lie down while inhaling.** Stop inhalation if you feel unwell. **6** After completing the inhalation period recommended by your doctor, switch the ON/OFF switch **3** to position «**O**» to turn off the device and unplug it from the socket. **7** Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting».

CLEANING AND DISINFECTING

Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. The compressor **1** and the air hose **6** should be cleaned with a clean, moist cloth.

 Always wash your hands well before cleaning and disinfecting the accessories.

 Do not expose the compressor to water or heat.

 Replace the air hose for each treatment with a new patient or in case of impurities.

 **Unplug the device before cleaning.**

Cleaning with water

Wash all nebuliser components (except air hose) under warm tap water (max 60 °C) for about 5 minutes adding if necessary a small quantity of detergent following dosage and use limitations as provided by detergent manufacturer. Rinse thoroughly making sure that all deposits are washed away and leave to dry.

Disinfecting

All nebuliser components (except air hose) can be disinfected with chemical disinfectants following dosage and use limitations as provided by disinfectant manufacturer. Disinfectants are usually available at pharmacies.

Sterilizing with steam

All nebuliser components (except air hose and masks) can be heat steam sterilized up to 121 °C (20 min.) or 134 °C (7 min.). EN554/ISO11134. The sterilization packaging must conform to EN868/ISO11607 and be suitable for steam sterilization. After sterilization always let all components cool down to ambient temperature before further use. Do not repeat sterilization cycle when components are still warm.

MAINTENANCE, CARE, AND SERVICE

Order all spare parts from your dealer or pharmacist.

It is advisable to replace the nebulizer after some 100 to 120 treatments on single patient or after about 20 sterilization cycles. Check the filter continually for cleanliness and replace it if dirty, or after a maximum of 3 months use. Spare filters are provided with the device. To replace the filter, open the air filter compartment **4** marked «Filter» and change the filter **11**.

MALFUNCTIONS AND ACTIONS TO TAKE

The device cannot be switched on

Ensure the power cord **2** is correctly plugged into the socket. Ensure the ON/OFF switch **3** is in the position «I». The device has automatically switched off due to overheating. Wait until it has cooled down and try again.

The nebuliser functions poorly or not at all

Ensure the air hose **6** is correctly connected at both ends. Ensure the air hose is not squashed, bent, dirty or blocked. If necessary, replace with a new one. Ensure the nebuliser **5** is correctly assembled **12**. Ensure the required medication has been added.

TECHNICAL DATA

Piston electrocompressor with thermal cut

Nominal voltage: see data label

Dimensions: 130x190x100 mm

Weight: 1.1 Kg approx.

Temporary use: 20/40 min I/O

MDD 93/42/EEC - Class of risk IIa

Maximum pressure: 185 kPa

Air flow: 10 l/min

Noise level: 58 dBA (EN13544-1)

Neb rate: 0.35 ml/min (con soluzione NaCl 0.9%)

Operating pressure: 60 kPa (0.6 bar)

Operating flow: 5 l/min

MMAD: 2.61 µm GSD: 2.75 Aerosol Output: 143 µl Output Rate: 87 µl/min

For any improvement reasons, Beper reserves the right to modify or improve the product without any notice.

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CE 0434

The European directive (Dir. 2012/19/UE-WEEE) on Waste Electrical and Electronic Equipment (WEEE), requires that old household electrical appliances must not be disposed of in the normal unsorted municipal waste stream. Old appliances must be collected separately in order to optimize the recovery and recycling of the materials they contain, and reduce the impact on human health and the environment. The crossed out "wheeled bin" symbol on the product reminds you of your obligation, that when you dispose of the appliance, it must be separately collected.

GUARANTEE CERTIFICATE

This appliance has been checked in the factory. From the date of original purchase a 24 month guarantee applies to material and production defects. The purchase receipt and the guarantee certificate must be submitted together in case of claim to guarantee.

For any technical assistance, please contact directly the seller or our head office in order to preserve the appliance's efficiency and NOT TO void the guarantee. Any intervention on this appliance by non-authorized persons will automatically void the guarantee.

Guarantee conditions

If the appliance shows defects as a result of faulty material and/or production during the period of guarantee, we guarantee repair free of charge on condition that:

- The appliance has been used properly and for the purpose for which it has been intended.
- Repairs are professionally carried out by qualified persons who are appointed by the supplier.
- The purchase receipt shall be presented.
- The appliance showing a fair wear and tear shall not be covered by this guarantee.

Therefore, any part that could be accidentally broken or having visible signs of use in consumable products (such as lamps, batteries, heating elements...) is excluded from the guarantee, and any defect whatsoever resulting from non-respect of the rules for use, negligence in usage and/or maintenance of the appliance, damage during transport and any other damage not attributable the supplier.

For every defect that could not be repaired within the guarantee period, the appliance will be replaced free of charge.

Technical assistance

Even after the guarantee period we will always pay attention to repairs for defected appliances. For technical assistance and/or repairs after the guarantee period you can directly contact the below address:

CONTACT YOUR DISTRIBUTOR IN YOUR COUNTRY OR AFTER SALES DEPARTMENT
BEPER. E-MAIL assistenza@beper.com
WHICH WILL FORWARD YOUR ENQUIRIES TO YOUR DISTRIBUTOR.

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