- MISURATORE DI PRESSIONE DA BRACCIO - MANUALE DI ISTRUZIONI
  - ARM BLOOD PRESSURE MONITOR - USE INSTRUCTIONS
- MESUREUR DE LA TENSION DU BRAS - MANUEL D'INSTRUCTIONS
  - OBERARM-BLUTDRUCK-MESSGERÄT - BETRIEBSANLEITUNG
  - TENSIÓMETRO DE BRAZO - MANUAL DE INSTRUCCIONES

CE 0197

LOT 2013-120

ITALIANO	.....	pag. 3
ENGLISH	.....	pag. 16
FRANÇAIS	.....	pag. 28
DEUTSCH	.....	pag. 41
ESPAÑOL	.....	pag. 53

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e-mail: assistenza@beper.com

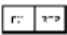
### Safety Notice

Thank you for purchasing the BP-1318 Blood Pressure Monitor. The BP-1318 Blood Pressure Monitor uses the oscillometric method of blood pressure measurement to measure the systolic, diastolic and pulse rate of an adult; Please read this manual thoroughly before using the unit. Please retain this manual for future reference. For specific information about your blood pressure, please CONSULT YOUR DOCTOR. To avoid risk and damage follow all warning precautions. Operate unit only as intended. Read all instructions prior to use


 <b>Caution</b>	
Individuals with serious circulation problems may experience discomfort. Consult your physician prior to use.	
Contact your physician if test results regularly indicate abnormal readings. Do not attempt to self-treat these symptoms without consulting your physician first.	
Product is designed for its intended use only. Do not misuse in any way.	
Product is not intended for infants or individuals who cannot express their intentions.	
Do not disassemble or attempt to repair.	
Do not use cell phones and other devices, which generate strong electrical or electromagnetic fields, near the device, as they may cause incorrect readings.	
Only use a recommended AC adaptor complying with EN 60601-1 and EN 60601-1-2 (see page 6). An unauthorized adaptor may cause fire and electric shock.	

 <b>Battery Precautions</b>	
Do not mix new and old batteries simultaneously.	
Replace batteries when <i>Low Battery Indicator</i> "  " appears on screen.	
Be sure battery polarity is correct.	
Do not mix battery types. Long-life alkaline batteries are recommended.	
Remove batteries from device when not in operation for more than 3 months.	
Dispose batteries properly; observe local laws and regulations.	


WARNING SIGNS AND SYMBOLS USED	
	Manufacturer
	Authorized Representative in the European Community
	Caution
	Mandatory
	Prohibited
	Type B Applied Part

### Monitor Unit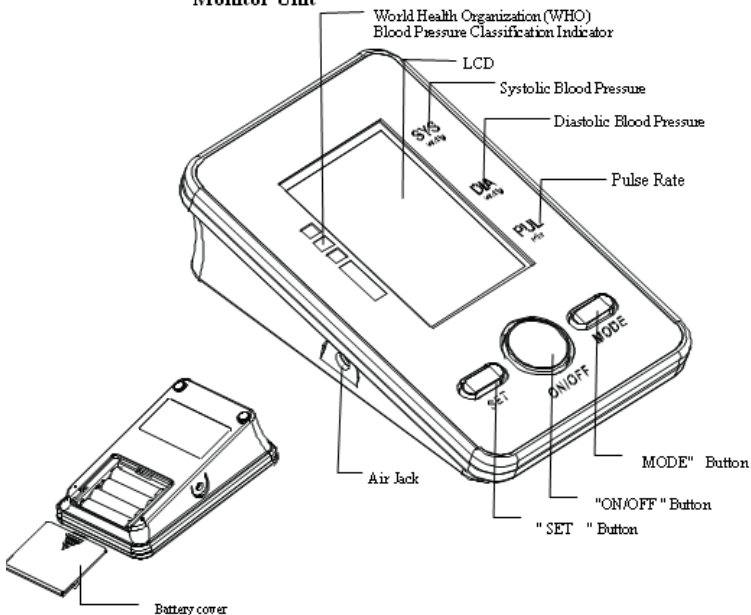

## Display

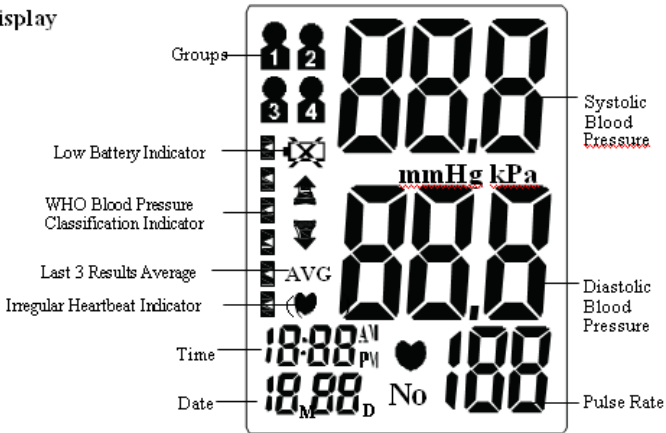
### Important Testing Guidelines

1. Avoid eating, exercising, and bathing for 30 minutes prior to testing.
2. Sit in a calm environment for at least 5 minutes prior to testing.
3. Do not stand while testing. Sit in a relaxed position while keeping your arm level with your heart.
4. Avoid speaking or moving body parts while testing.
5. While testing, avoid strong electromagnetic interference such as microwave ovens and cell phones.
6. Wait 3 minutes or longer before re-testing.
7. Try to measure your blood pressure at the same time each day for consistency.
8. Test comparisons should only be made when monitor is used on the same arm, in the same position, and at the same time of day.
9. This blood pressure monitor is not recommended for people with severe arrhythmia.

### Quick Start

1. Install batteries. (See Figure A)
2. Insert cuff air plug into the left side of monitor unit. (See Figure B)
3. Remove thick clothing from the arm area.
4. Rest for several minutes prior to testing. Sit down in a quiet place, preferably at a desk or table, with your arm resting on a firm surface and your feet flat on the floor. (See Figure C)
5. Apply cuff to your left arm and keep level with your heart. Bottom of cuff should be placed approximately 1-2cm (0.4-0.8") above elbow joint. (See Figures D&E)
6. Press " ON/OFF " Button to start testing.

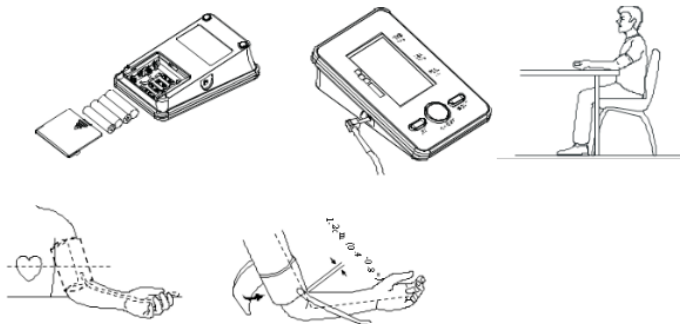### Battery Installation

Slide battery cover off as indicated by arrow. Install 4 new AAA alkaline batteries according to polarity. Close battery cover.

### System Settings

With power off, press "SET" button to activate System Settings. The Memory Group icon flashes.

#### 1. Select Memory Group

While in the System Setting mode, you may accumulate test results into 2 different groups. This allows multiple users to save individual test results (up to 60 memories per group.) Press "MODE" button to choose a group setting. Test results will automatically store in each selected group.

#### 2. Time /Date Setting

Press "SET" button again to set the Time/Date mode. Set the month first by adjusting the "MODE" button. Press "SET" button again to confirm current month. Continue setting the day, hour, and minute in the same fashion. Every time the "SET" button is pressed, it will lock in your selection and continue in succession (month, day, hour, minute.)

#### 3. Saved Settings

While in any setting mode, press "ON/OFF" button to turn the unit off.

All information will be saved.

Note: If unit is left on and not in use for 3 minutes, it will automatically save all information and shut off.

### Applying the Arm Cuff

1. Firmly insert air plug into opening located on left side of monitor unit.
2. With sticky nylon section facing outward, insert end of cuff underneath metal ring of cuff
3. Fasten cuff about 1-2cm (0.4-0.8") above the elbow joint. For best results apply cuff to bare arm and keep level with heart while testing.

Note: Do not insert air plug into opening located on right side of monitor unit.

This opening is designed for an optional power supply only.

## Testing

### 1. Power On

Press and hold " ON/OFF " button until a beep sounds. The LCD screen will appear for one second as unit performs a quick diagnosis. A long tone indicates device is ready for testing.

Note: Unit will not function if residual air from previous testing is present in cuff.

The LCD will flash until pressure is stabilized

### 2. Pressurization

Initial pressure is first pumped to 190 mmHg. If the current user's systolic blood pressure is over 190mmHg, the unit will automatically re-inflate to the proper shelf.

Note: Pressurization will gradually subside and ultimately stop when cuff is not properly applied to the arm. If this occurs, press " ON/OFF "button to turn the unit off.

### 3. Testing

After cuff inflation, air will slowly subside as indicated by the corresponding cuff pressure value. A flashing will appear simultaneously on screen signaling heart beat detection.

Note: Keep relaxed during testing. Avoid speaking or moving body parts.

### 4. Result Display

Three short beeps sound when testing is complete. The screen will display measurements for systolic and diastolic blood pressure. A indicator representing the current measurement will appear next to the corresponding WHO Classification.

Note: Refer to Page 23~24 for detail WHO Blood Pressure Classification Information.

## Irregular Heartbeat Indicator

If the monitor detects an irregular heart rhythm two or more times during the measuring process, the Irregular Heartbeat Symbol appears on screen along with measurement results. Irregular heartbeat rhythm is defined as rhythm that is either 25%

slower or faster than the average rhythm detected while measuring systolic blood pressure and diastolic blood pressure. Consult your physician if the Irregular Heartbeat Symbol frequently appears with your test results.

## 5. Deleting/Storing Test Results

User may delete their current test result due to unfavorable testing conditions or for any other reason. To delete the last test result, press the "SET" button after result is displayed.

If result is not deleted, it will automatically store by date within the previously configured Memory Group.

Note: Be sure the appropriate Memory Group selection is made prior to testing.

If the number of tests surpasses the allotted 60 memories per group, the most recent tests will appear first, thus eliminating the oldest readings.

## Power Off

The " ON/OFF " button can be pressed to turn off the unit in any mode. The unit can turn off the power itself about 3 minutes no operation in any mode.

**Safety Precaution:** If pressure in arm cuff becomes too extreme while testing, press the “ ON/OFF “ button to turn power off. The cuff pressure will rapidly dissipate once the unit is off.

### **Memory Check**

With power off, you may check past test results by using the “ MODE “ buttons. The most recent test result and oldest test result in memory can be viewed by pressing and holding the “ MODE “ button upon activating test results you can press the “ MODE “ button to scroll through all test results stored in memory.

**Note:** Previous test results will only be displayed from the most recently used memory group. To check previous test results in other memory groups, you must first select the desired group and then turn monitor off. (See Select Memory Group on Page 10.)

### **Memory Deletion**

Memory for a selected group may be deleted while in Memory Check mode. Press and hold the “ SET “ button for approximately 3 seconds to delete all memory records from the selected group. The monitor will beep indicating successful deletion and then transfer into testing mode. Press the “ ON/OFF “ button to turn the unit off.

**Note:** Memory cannot be recovered once it has been deleted.

### **Last 3 Tests Average**

With power off, press and hold the “ MODE “ button to activate screen display. After the unit performs a self diagnosis, the screen will display the average test results from the last 3 readings of the last group used. The symbol will appear along with the corresponding WHO Blood Pressure Indicator. The Memory Check mode can be accessed by pressing the “MODE “ buttons. To check the average results from other groups, select the desired group first prior to activating the “ MODE “ button in the off position.

### **Low Battery Indicator**

4 short warning beeps sound when battery life is depleting and unable to inflate cuff for testing. The “  “ appears simultaneously for approximately 5 seconds prior to shutting off. Replace batteries at this time. No memory loss will occur throughout this process.

**Troubleshooting**

<b>PROBLEM</b>	<b>CAUSE</b>	<b>SOLUTION</b>
Blood pressure results are not within typical range	Cuff is too tight or not properly positioned on the arm	Firmly reposition cuff approximately above the elbow joint ( See Page 12)
	Inaccurate test results due to body movement or monitor movement	Sit in a relaxed position with arm placed near heart. Avoid speaking or moving body parts while testing. Make sure the monitor unit is placed in a stationary position throughout the testing period. ( See Page 7)
“ Err” displayed	Cuff fails to inflate properly	Make sure hose is properly fastened to cuff and monitor unit
	Improper operation	Read user manual carefully and re-test properly.
	Pressurization is over 300mmHg	Read user manual carefully and re-test properly.

**Blood Pressure Information****Blood Pressure**

Blood pressure is the force of blood pushing against the walls of arteries. It is typically measured in millimetres of mercury (mmHg.) Systolic blood pressure is the maximum force exerted against blood vessel walls each time the heart beats. Diastolic blood pressure is the force exerted on blood vessels when the heart is resting between beats.



An individual's blood pressure frequently changes throughout the course of a day.

Excitement and tension can cause blood pressure to rise, while drinking alcohol and bathing can lower blood pressure. Certain hormones like adrenaline (which your body releases under stress) can cause blood vessels to constrict, leading to a rise in blood pressure.

If these measuring numbers become too high, it means the heart is working harder than it should.

**WHO Blood Pressure Classification Indicator**

The BP-1318 is equipped with a classification indicator based on established guidelines from the World Health Organization. The chart below (color coded on monitor unit) indicates test results.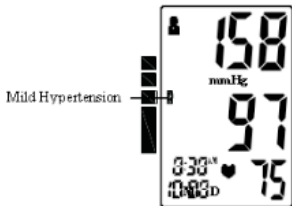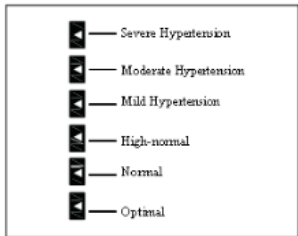

### Health Reminder

Hypertension is a dangerous disease that can affect the quality of life. It can lead to a lot of problems including heart failure, kidney failure, and cerebral hemorrhaging.


By maintaining a healthy lifestyle and visiting your physician on a regular basis, hypertension and relative diseases are much easier to control when diagnosed in their early stages.

**Note:** Do not be alarmed if an abnormal reading occurs. A better indication of an individual's blood pressure occurs after 2-3 readings are taken at the same time each day over an extended period of time. Consult your physical if test results remain abnormal.

### Blood Pressure Q&A

**Q:** What is the difference between measuring blood pressure at home or at a professional healthcare clinic?

**A:** Blood pressure readings taken at home are now seen to give a more accurate account as they better reflect your daily life. Readings can be elevated when taken in a clinical or medical environment. This is known as White Coat Hypertension and may be caused by feeling anxious or nervous.

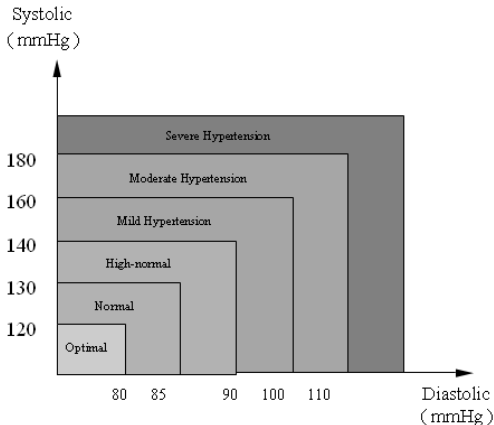## Arm blood pressure monitor Use Instructions

**Note:** Abnormal test results may be caused by:

1. Improper cuff placement  
Make sure cuff is snug-not too tight or too loose.  
Make sure bottom of the cuff is approximately 1-2cm (1/2") above the elbow joint.
2. Improper body position  
Make sure to keep your body in an upright position.
3. Feeling anxious or nervous  
Take 2-3 deep breaths, wait a few minutes and resume testing.

Q: What causes different readings?

A: Blood pressure varies throughout the course of a day. Many factors including diet, stress, cuff placement, etc. may affect an individual's blood pressure.

Q: Should I apply the cuff to the left or right arm? What is the difference?

A: Either arm can be used when testing, however, when comparing results, the same arm should be used. Testing on your left arm may provide more accurate results as it is located closer to your heart.

Q: What is the best time of day for testing?

A: Morning time or any time you feel relaxed and stress free.

## Maintenance

1. Avoid dropping, slamming, or throwing the unit.
2. Avoid extreme temperatures. Do not expose unit directly under sunshine.
3. When cleaning the unit, use a soft fabric and lightly wipe with mild detergent.  
Use a damp cloth to remove dirt and excess detergent.
4. Cuff Cleaning: Do not soak cuff in water! Apply a small amount of rubbing alcohol to a soft cloth to clean cuff's surface. Use a damp cloth (water-based) to wipe clean. Allow cuff to dry naturally at room temperature.
5. Do not use petrol, thinners or similar solvents.
6. Remove batteries when not in operation for an extended period of time.
7. Do not disassemble product.
8. It is recommended the performance should be checked every 2 years.

## Specifications

## Arm blood pressure monitor Use Instructions

Product Description	Arm-type Fully Automatic Digital Blood Pressure Monitor	
Model	BP-1318	
Display	LCD Digital Display Size: 66mm 43mm (3.31" x 2.17")	
Measurement Method	Oscillometric Method	
Measurement Range	Pressure	0mmHg 300mmHg
	Pulse	30 to 180 Beats/Minute
Measurement Accuracy	Pressure	3mmHg
	Pulse	5%
Pressurization	Automatic Pressurization	
Memory	120 Memories in Two Groups with Date and Time	
Function	Irregular Heartbeat Detection	
	WHO Classification Indicator	
	Last 3 Results Average	
	Low Battery Detection	
	Automatic Power-Off	
Power Source	4 AAA	
Battery Life	Approximately 2 months at 3 tests per day	
Unit Weight	Approx.405g (14.30 oz.) (excluding battery)	
Unit Dimensions	Approx.139 x 88 x 42mm (4.33" x 6.10" x 2.76" )(L x W x H)	
Cuff Circumference	22.0cm 36.0 cm (8.6" ~ 14.2")	
Operating Environment	Temperature	10□ ~ 40□ (50□~104□)
	Humidity	15% ~ 90% RH
Storage Environment	Pressure	-20 ~55 (-4□~131□)
	Temperature:	15% ~ 90% RH
	Humidity	525mmHg 795mmHg
	Pressure	

This device complies with the provisions of EC directive 93/42/EEC (Medical Device Directive.) It is designed according to the European Standard EN1060, Noninvasive sphygmomanometers Part 1: General Requirements and Part 3: Supplementary requirements for electromechanical blood pressure measuring systems.

**For any improvement reasons, Beper reserves the right to modify or improve the product without any notice.**

## DISPOSAL OF OLD ELECTRICAL APPLIANCES

The European directive 2011/65/EU on Waste Electrical and Electronic Equipment (WEEE), requires that old household electrical appliances must not be disposed of in the normal unsorted municipal waste stream. Old appliances must be collected separately in order to optimize the recovery and recycling of the materials they contain, and reduce the impact on human health and the environment.

The crossed out “wheeled bin” symbol  on the product reminds you of your obligation, that when you dispose of the appliance, it must be separately collected.

## **GUARANTEE CERTIFICATE**

---

---

This appliance has been checked in the factory. From the date of original purchase a 24 month guarantee applies to material and production defects. The purchase receipt and the guarantee certificate must be submitted together in case of claim to guarantee.

For any technical assistance, please contact directly the seller or our head office in order to preserve the appliance's efficiency and NOT TO void the guarantee. Any intervention on this appliance by non-authorized persons will automatically void the guarantee.

### **Guarantee conditions**

If the appliance shows defects as a result of faulty material and/or production during the period of guarantee, we guarantee repair free of charge on condition that:

- The appliance has been used properly and for the purpose for which it has been intended.
- Repairs are professionally carried out by qualified persons who are appointed by the supplier.
- The purchase receipt shall be presented.
- The appliance showing a fair wear and tear shall not be covered by this guarantee.

Therefore, any part that could be accidentally broken or having visible signs of use in consumable products (such as lamps, batteries, heating elements...) is excluded from the guarantee, and any defect whatsoever resulting from non-respect of the rules for use, negligence in usage and/or maintenance of the appliance, damage during transport and any other damage not attributable the supplier.

For every defect that could not be repaired within the guarantee period, the appliance will be replaced free of charge.

### **Technical assistance**

Even after the guarantee period we will always pay attention to repairs for defected appliances. For technical assistance and/or repairs after the guarantee period you can directly contact the below address:

---

---

CONTACT YOUR DISTRIBUTOR IN YOUR COUNTRY OR AFTER SALES DEPARTMENT  
BEPER. E-MAIL [assistenza@beper.com](mailto:assistenza@beper.com)  
WHICH WILL FORWARD YOUR ENQUIRIES TO YOUR DISTRIBUTOR.

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